



ReSound LiNX 3D 61-DRWZ ZPower Battery Door Instructions


1. To remove the existing battery door, open the battery door and pull straight out.

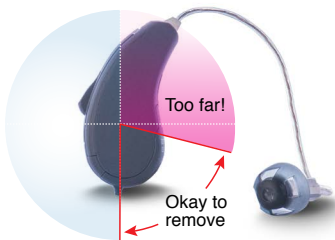

2. Remove the hinge pin from the existing battery door with a small tweezers or something similar. Keep the hinge pin!

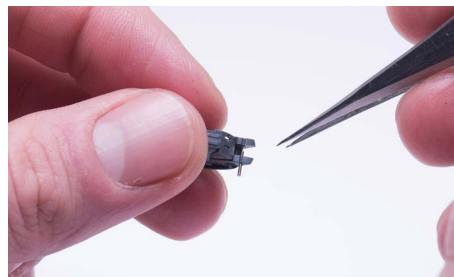


3. Install the same hinge pin into the ZPower battery door.

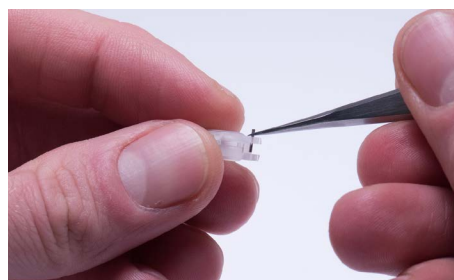

4. Snap the ZPower battery door into place by pushing straight in until you hear and/or feel a click.



5. Insert a new ZPower silver Zinc battery and place the hearing instrument into the charger. Allow up to four hours for a full charge.

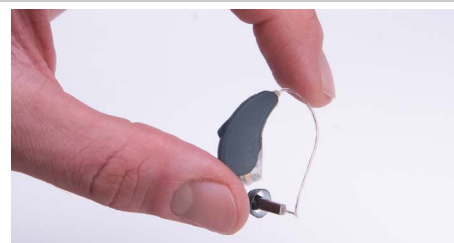

Manufacturer according to FDA:

ReSound North America

8001 E. Bloomington Freeway
Bloomington, MN 55420
1.800.248.4327
resoundpro.com

ReSound Government Services

8001 E. Bloomington Freeway
Bloomington, MN 55420
1.800.392.9932
resound.com/governmentservices

Manufacturer according to Health Canada:

ReSound Canada

303 Supertest Road
Toronto, Ontario M3J 2M4
1.888.737.6863
resoundpro.com



MK604942 Rev B 2017.10

ReSound GN