


Tech Tips

ReSound GN


Smart Fit | Custom Device Tone Indicators

In Smart Fit 1.1, indicator tones may be missing in Fitting > Device Controls for LiNX 3D custom devices (as shown below).

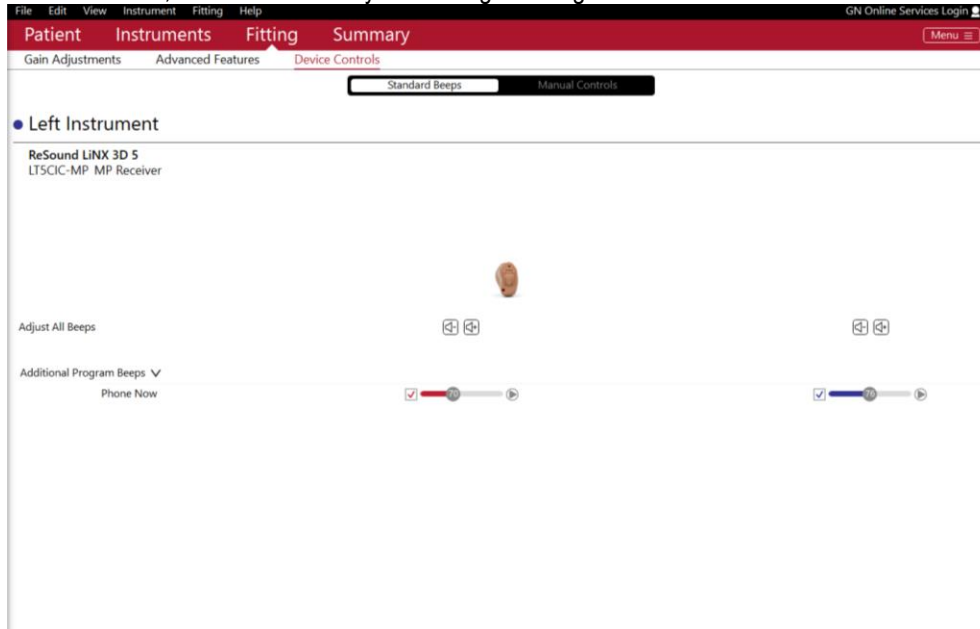

Workaround:

If indicator tones are missing, click on any other tab in the red header (i.e. Patient, Instruments, or Summary), then click back to Fitting > Device Controls. All options will now be present.

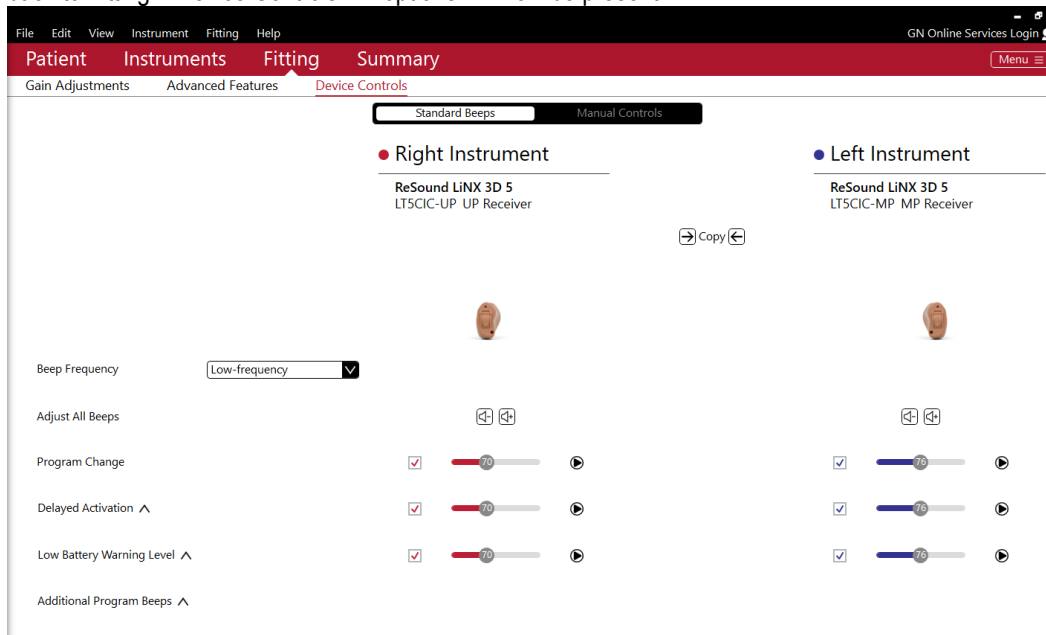

Manufacturer according to FDA:

ReSound North America
8001 E. Bloomington Freeway
Bloomington, MN 55420
800.248.4327
resoundpro.com

Manufacturer according to Health Canada:

ReSound Canada
303 Supertest Road
Toronto, Ontario M3J 2M4
1.888.737.6863
resoundpro.com

