

ReSound Up Smart™

Because every word matters

ReSound Up Smart™ is a premium hearing aid family for children of all ages and hearing losses. It is a game changer in the way children are granted easy and direct access to the sounds they need for speech and language development. ReSound Up Smart meets or surpasses all pediatric requirements, including safety requirements.

Surround Sound by ReSound™ ensures audibility and consistently clear and comfortable sound, even at high output levels, in all listening situations, due to advanced pediatric features.

As the first pediatric Made for iPhone hearing solution, ReSound Up Smart 9, 7 and 5 receives stereo sound directly from an iPhone, iPad or iPod touch to benefit listening and language learning in more fun ways for infants and pre-schoolers, and to provide educational and social media interaction possibilities for older children and teens. And with the ReSound Smart™ app, the child or the parent can adjust the hearing aids easily, discreetly and intuitively from iOS devices.

ReSound Up Smart offers a line-up of modern and powerful wireless BTEs for mild to profound hearing losses.

Product features

- Wireless connectivity to ReSound accessories
- Wireless ear-to-ear connectivity
- Full iSolate™ nanotech coating
- Supports open and closed configurations
- Push button and programmable Volume Control
- Size 13 battery
- Battery door with integrated On/Off switch
- Advanced, auto-locking battery door
- LED visual indication
- Telecoil with T and MT modes

Options

- 12 different color combinations
- Direct audio input or integrated FM
- Supports Dome, Tulip-Dome and Standard Earmold
- Standard hook, Child hook, Baby hook and Thin Tube adaptor

Fitting software

- Aventa fitting software (3.10 or higher)
- Wireless fitting interface: Airlink™
- Programming adaptor with CS44 socket cable
- Speedlink, HI-Pro, or NOAHlink programming interfaces

UPS977-DLW
UPS777-DLW
UPS577-DLW
UPS577M-DLW




ReSound Up Smart 9
ReSound Up Smart 7
ReSound Up Smart 5

	ReSound Up Smart 9	ReSound Up Smart 7	ReSound Up Smart 5
ReSound SmartRange chip	●	●	●
Surround Sound by ReSound			
Model			
WARP™ Compression - number of bands	17	17	9
Environmental Classifier	●	●	●
Sound Shaper™	●	●	●
Clean			
NoiseTracker™ II	●	○	○
Personalized Noise Reduction	●		
WindGuard™	●	○	○
Expansion	●	○	○
Balance			
Binaural Directionality™	●		
Directional Mix Processor	●	●	●
- Adjustable directional mix	●		
Natural Directionality™ II	●	●	
Synchronized SoftSwitching™	●	●	
SoftSwitching™			●
AutoScope Adaptive Directionality™	●		
MutiScope Adaptive Directionality™	●	●	
Adaptive Directionality			●
Binaural Environmental Optimizer™ II	●		
Environmental Optimizer™ II	●		
Environmental Optimizer™		●	
Stabilize			
DFS Ultra™ II	●	●	●
- Music Mode™	●	●	●
Auto DFS™	●	●	●
Convenience Features			
Synchronized Push Button	●	●	●
Synchronized Volume Control	●	●	●
SmartStart™	●	●	●
PhoneNow™	●	●	●
Comfort Phone™	●	●	●
iSolate™ nanotech	●	●	●
Fitting Features			
Gain Handles	9	9	9
Fully Flexible Programs	4	4	4
Synchronized Acceptance Manager	●	●	●
Onboard Analyzer™ II	●	●	●
In Situ Audiometry	●	●	●
Wireless connectivity			
2.4 GHz Wireless Technology	●	●	●
2.4 GHz ear-to-ear communication	●	●	●
Direct audio streaming (Made for iPhone)	●	●	●*
Wireless fitting with Airlink™	●	●	●
ReSound Unite™ TV Streamer 2	●	●	●
ReSound Unite™ Remote Control 2	●	●	●
ReSound Unite™ Phone Clip+	●	●	●
ReSound Multi Mic and Micro Mic	●	●	●
ReSound Control™ app (Phone Clip+ required)	●	●	●
ReSound Smart™ app	●	●	●*

*Available in UPS577M-DLW

Fitting range

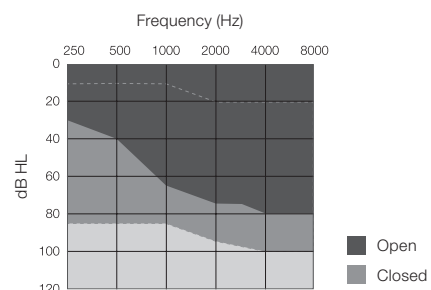

ReSound Up Smart is compatible with iPhone 6s Plus, iPhone 6s, iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPad Pro, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 4, iPad mini 3, iPad mini 2, iPad mini and iPod touch (5th generation) using iOS 7.X or later. Apple, the Apple logo, iPhone, iPad, iPad Air, iPad mini and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.



○ Standard
● Advanced
● Ultimate

ReSound Up Smart™


Technical specifications

		UPS77-DLW Open UPS77M-DLW Open		UPS77-DLW Closed UPS77M-DLW Closed	
		IEC 60118-0 IEC 711 Ear simulator	IEC 60118-7 ANSI S3.22 2cc coupler	IEC 60118-0 IEC 711 Ear simulator	IEC 60118-7 ANSI S3.22 2cc coupler


Reference test gain (60 dB SPL input)	1600 Hz/HFA	44	41	47	47	dB
Full-on gain (50 dB SPL input)	Max.	62	51	66	59	dB
	1600 Hz/HFA	53	46	56	52	
Maximum output (90 dB SPL input)	Max.	130	122	134	128	dB SPL
	1600 Hz/HFA	126	118	130	123	
Total harmonic distortion	500Hz	0.2	0.1	0.5	0.4	%
	800 Hz	0.5	0.2	0.6	0.4	
	1600 Hz	0.4	0.5	0.6	0.5	
Telecoil sensitivity (1 mA/m input)	Max.	91	-	94	-	dB SPL
Full-on Telecoil sensitivity @ 1mA/m	1600 Hz / HFA	81	75	84	81	
HFA – SPLIV @ 31.6 mA/m (ANSI)	HFA	-	99	-	106	
Equivalent input noise w/o Noise reduction		23	23	23	22	dB SPL
1/3 Octave Equivalent input noise, w/o Noise reduction		10		10		
Frequency range (DIN 45605/ANSI)		100-6900	100-6770	100-7080	100-6850	Hz
Current drain		1.1	1.2	1.1	1.2	mA

Data in accordance with IEC 60118-0, IEC 60118-7;
Supply Voltage 1.3 V.


Maximum Output (OSPL 90)

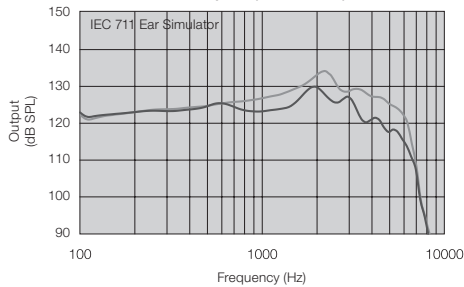


Maximum Output (OSPL 90)

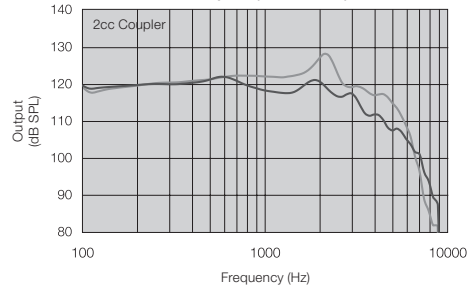


Full-On and Reference Test Gain

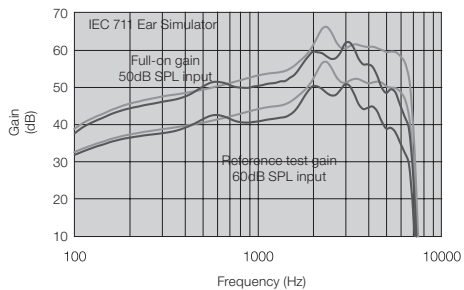


Full-On and Reference Test Gain

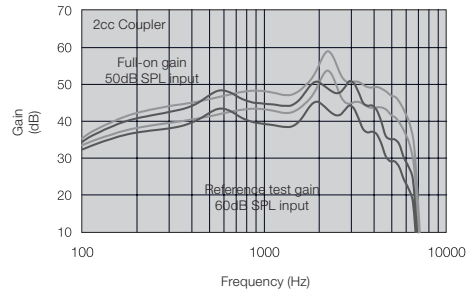

Telecoil response
Input level 10 mA/m

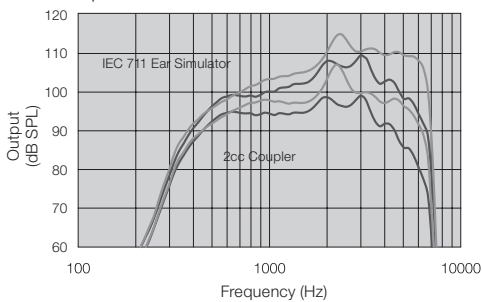

Input/Output Response

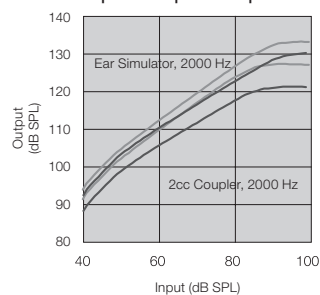

Measured according to IEC 60 118-0 1983, amendment 1994; at 1.3 V, impedance 6.2 ohms and 23°C on O.E.S. according to IEC711 1981, resp on 2cc according to IEC60118-7 2nd edition 2005 (DIN average calculated at 500 Hz, 1000 Hz and 2000 Hz; HFA average calculated at 1000 Hz, 1600 Hz and 2500 Hz; 0 dB SPL sound pressure equals 20µPa). All measurements without DSP features activated unless indicated otherwise.

- Open
- Closed

Basic settings:
Full-on Gain, Reference Test Gain
MPO = Maximum Peak Output
Maximum Band Width

Manufacturer according to FDA:

ReSound North America
8001 Bloomington Freeway
Bloomington, MN 55420
1-888-735-4327
resound.com

ReSound Government Services
8001 Bloomington Freeway
Bloomington, MN 55420
1-800-392-9932
resound.com/veterans

Manufacturer according to Health Canada:

ReSound Canada
303 Supertest Road
Toronto, Ontario M3J 2M4
1-888-737-6863
resound.com



Patents pending

All specifications are subject to change without notice

400310011-US-16.09-Rev.C